Impressive esthetics Beeindruckende Ästhetik

Dr Gabriel Krastl
Switzerland

Dr Andrea Shepperson
New Zealand

Dr Martin von Sontagh
Austria

Marshall Hanson
USA

Dr Jorge Eustáquio
Brazil

Jan Kurtz-Hoffmann
Germany

Dr Eduardo Mahn
Chile

Dr Markus Lenhard
Switzerland

Prof. Dr Antonio Cerutti
Italy

Dr Olivier Romieu
France

Dr Rafael Piñeiro Sande
Spain

Dr Tomasz Sosnowski
Poland

IPS Empress® Direct
Casebook

ivoclar
vivadent®
passion vision innovation

Dr Martin von Sontagh

“

I primarily use IPS Empress® Direct for esthetic and natural-looking resin restorations. Due to the wide array of shades available, teeth can be rebuilt layer by layer in a lifelike fashion. Moreover, IPS Empress Direct is particularly suitable for bruxing patients because of its high wear resistance.

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Case presentation

Many people process stress by grinding their teeth at night. Therefore, bruxism is a very frequently diagnosed disorder. Restorations with state-of-the-art composite resin such as IPS Empress Direct are often the treatment of choice in such cases, as they enable a minimally invasive restorative approach. Even patients with functional problems can be treated conservatively.

In the present case the patient wished the gap between his front teeth to be closed and the tissue defects to be restored. However, he declined orthodontic treatment. After having analyzed the photos and the model, a personalized treatment plan was designed. The tooth length and width proportions were newly defined and the axes of teeth 14 to 24 were reoriented by means of composite resin veneers.

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after | nachher

Starting situation: The functional problems and diastemata are obvious.

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A photographic analysis shows that the incisal line does not correspond with the natural smile line (1).

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The cranial view confirms that the tooth axes are not straight and the edges of the teeth do not follow the curve of the lip.

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Delicate veneer preparations are made to align the teeth along one axis.

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Then the teeth are built up in layers using different shades of IPS Empress Direct.

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The composite resin is shaped and adapted with OptraSculpt® Pad until a smooth surface is achieved.

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The reference scales on the OptraSculpt Pad handle assist in determining the tooth width.

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Result after completion of the treatment

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A realignment of the incisal line with the smile line has been achieved, which ensures a more natural look.

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